

Drink Spike Detector

By Drink Safe Technologies™

Think Before You Drink

A simple test that can be used to check if a drink may have been tampered with using GHB or Ketamine.

Product Information

What is the Drink Spike Detector?

The Drink Spike Detector by Drink Safe Technologies™ is a litmus style test that can be used to check if a drink may have been tampered with by the addition of GHB or Ketamine.

How does it work?

Note that there are two tests on each card.

- Place a drop from the suspect drink onto both spots of one test.
- Wait a couple of minutes for it to dry, and look for a colour change.
- If a darker blue colour is visible, this is a positive result and we advise to discard the drink.

Ensure that the results are read in a well lit area.

The test results are intended for personal use and informational purposes only.



1. PLACE A DROP OF YOUR BEVERAGE ONTO BOTH SPOTS OF ONE TEST. SMEAR EACH GENTLY



2. WAIT A FEW MINUTES UNTIL DRY. **RESULTS ARE BEST VIEWED IN A WELL LIT AREA.**



3. IF EITHER SPOT TURNS TO A DARKER BLUE COLOUR, ONE OR MORE DRUGS WHICH COULD BE USED IN DRUG FACILITATED CRIME MAY HAVE BEEN DETECTED.

What drugs are detected?

- GHB (also known as fantasy, liquid E, GBH, grievous bodily harm)
 - Ketamine (also known as K, special K)
- These are considered to be the two most common illicit drugs used to spike drinks.

How do I avoid being a victim?

Reports of drink spiking are increasing nationally and it is suggested that 1 in 4 victims of sexual assault may have had their drink spiked.

- Watch your drink being prepared and at all times, and don't accept drinks from strangers.
- Don't leave your drink unattended.
- Don't share drinks
- Don't drink a drink that doesn't look, smell or taste right
- Look out for your friends & don't go out alone.
- Use a Drink Spike Detector if in any doubt about a drink

What are its limitations?

The Drink Spike Detector tests for GHB and Ketamine only, and will therefore not pick up every drug that could be placed into a beverage.

The test should be used in a well lit area.

The test results are intended for personal use and informational purposes only.

The product should not be used in the following beverages:

- Wine products
- Beverages containing milk or cream
- Oily liqueurs
- Beverages containing tonic water
- Beverages containing fruit juice
- Blue beverages

For further information

www.drinksafetech.com.au

Phone 1300 78 SAFE or 03 9387 7444

Packaged and Distributed in Australia by Drink Safe Technologies, 60 Brunswick Road, Brunswick, Melbourne, Victoria 3056

Drink Spike Detector by Drink Safe Technologies™ is a registered trademark of Drink Safe Technologies (Aust) P/L